# Regard®

Innovative  
contact lens solution

(EN)

Congratulations on choosing REGARD®, a scientific advancement in contact lens care that provides superior lens comfort and hygiene. REGARD contains an active disinfectant (complexed stabilized OxyChlorite® molecule) that breaks down into the components of natural tears (H<sub>2</sub>O and NaCl) and releases oxygen. It's kind to your eyes due to the absence of preservatives like PHMB.

REGARD cleans, rinses, disinfects, stores and lubricates soft contact lenses and is designed to help prevent the formation of protein deposits on the lens surface without enzyme protein removers.

REGARD is a sterile, isotonic, buffered solution that contains, purified water, sodium chloride, boric acid, hydroxypropylmethylcellulose (0,15%), poloxamer and OxyChlorite®.

## Instructions for use

- Always wash and rinse your hands thoroughly before removing your lenses.
- Place a few drops of REGARD solution on each contact lens and gently rub to clean the surface of your lenses.
- Rinse each lens with REGARD solution to remove debris. Never rinse your lenses with tap water.
- Only use the original REGARD lens case. Unscrew the cover corresponding to the lens that you want to store (cover with a heart for the lens for the left eye, or without a heart if it is the lens for the right eye) and fill the compartment to the brim with the solution REGARD.
- Store the lens depositing it in the compartment so that the corresponding outer side is oriented towards the bottom. Also make sure that the lens is placed in a way that it is not damaged when you close the lens case.
- Close tightly and turn the lens case upside down and back.
- Soak your lenses for at least six hours to disinfect your lenses and remove protein deposits. REGARD destroys harmful micro-organisms and their toxins and enzymes. If you store your lenses more than thirty days, repeat the disinfection with fresh REGARD solution.
- REGARD solution may be used to rewet and lubricate lenses before placing on your eyes.
- Rinse your lens case with fresh REGARD solution and allow to air dry after each use.

## Precautions

Bottle: Not use the product if the bottle is damaged or the seal is tampered.

Unidose: Do not use if the vial is not integral.

REGARD solution should never be reused.

Bottle: Avoid touching the bottle tip and close tightly after each use.

Unidose: Avoid touching the tip of the unit dose.

Store at room temperature and away from direct sunlight.

Do not use after expiration date.

Bottle: Discard an opened bottle after 90 days.

Unidose: The unit dose should never be reutilized after opening because it could become contaminated.

Always use the new lens case provided with the REGARD solution.

If you can't clean your lens case or in the presence of deformations, replace it with a new one.

As with any personal care product, discontinue if irritation occurs or you are allergic to any of the ingredients. To consult a specialist. Keep away from children. Not for internal use.

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